


FLUSH MOUNT DOOR DETAIL BRUNO VERTICAL PLATFORM LIFT

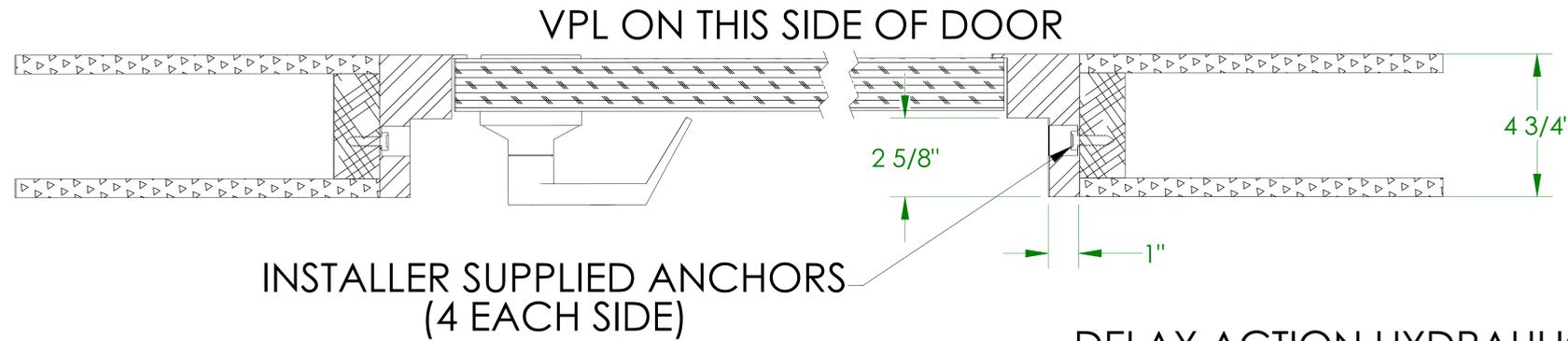


PART NO.	DESCRIPTION	'A'	'B'
VPL-32236	FLUSH MOUNT 36" OAK DOOR (LH SWING)	36"	41"
VPL-32237	FLUSH MOUNT 36" OAK DOOR (RH SWING)	36"	41"
VPL-32238	FLUSH MOUNT 36" STEEL DOOR (LH SWING)	36"	41"
VPL-32239	FLUSH MOUNT 36" STEEL DOOR (RH SWING)	36"	41"
VPL-32272	FLUSH MOUNT 46" OAK DOOR (LH SWING)	46"	51"
VPL-32273	FLUSH MOUNT 46" OAK DOOR (RH SWING)	46"	51"
VPL-32274	FLUSH MOUNT 46" STEEL DOOR (LH SWING)	46"	51"
VPL-32275	FLUSH MOUNT 46" STEEL DOOR (RH SWING)	46"	51"

DELAY ACTION HYDRAULIC CLOSER

KEYED HANDLE

NOTES:

- DOOR FRAME IS SHIPPED WITH TACKPLATE SILL WHICH IS REMOVED DURING INSTALLATION (USE A HAMMER TO REMOVE).
- INTERLOCK IS SHIPPED SEPARATELY TO PROTECT INTERLOCK. INSTRUCTIONS ARE INCLUDED WITH THE INTERLOCK FOR INSTALLATION INTO DOOR FRAME.
- FINISH:
 - OAK DOOR IS PLAIN (NOT FINISHED) AND FRAME IS GRAY PRIMER FINISH.
 - STEEL DOOR AND FRAME HAS A GRAY PRIMER FINISH.
- IF DOOR IS USED WITH A FIXED RAMP, RAISE THE DOOR FRAME UP THE HEIGHT OF THE FIXED RAMP. PLEASE ALLOW FOR THIS IN THE ROUGH OPENING IF NEEDED. YOU WILL HAVE TO FINISH THE BOTTOM OF THE DOOR FRAME AS NEEDED.
- ADD 1/2" TO OVERALL FRAME DIMENSIONS TO DETERMINE ROUGH OPENING.

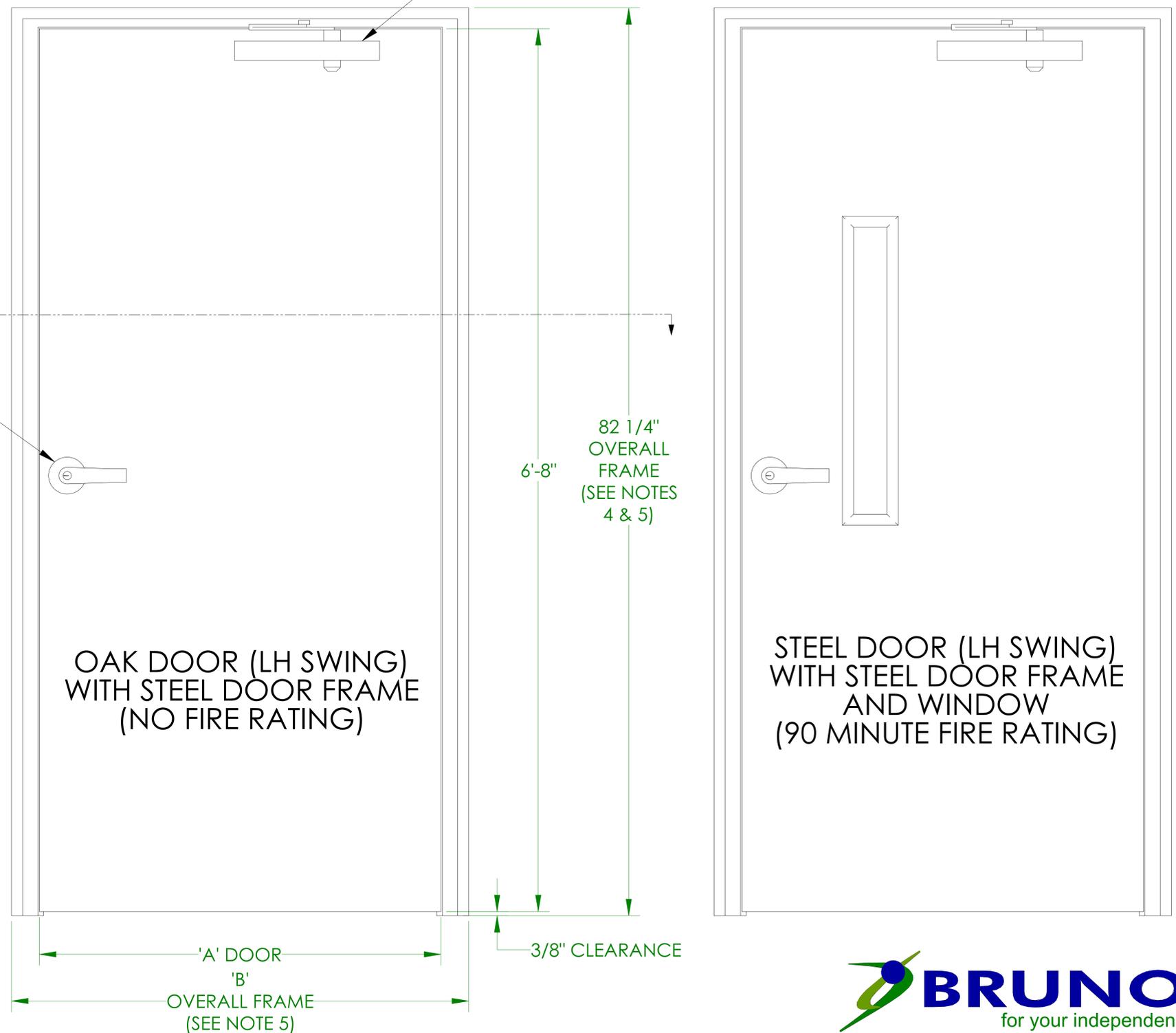


ILS-01027
SHEET 1 OF 4
REV. 3 (5917)(6/29/15)(DPG)



©Bruno Independent Living Aids, Inc.


FLUSH MOUNT DOOR ALIGNMENT (STRAIGHT-THRU PLATFORM) BRUNO VERTICAL PLATFORM LIFT

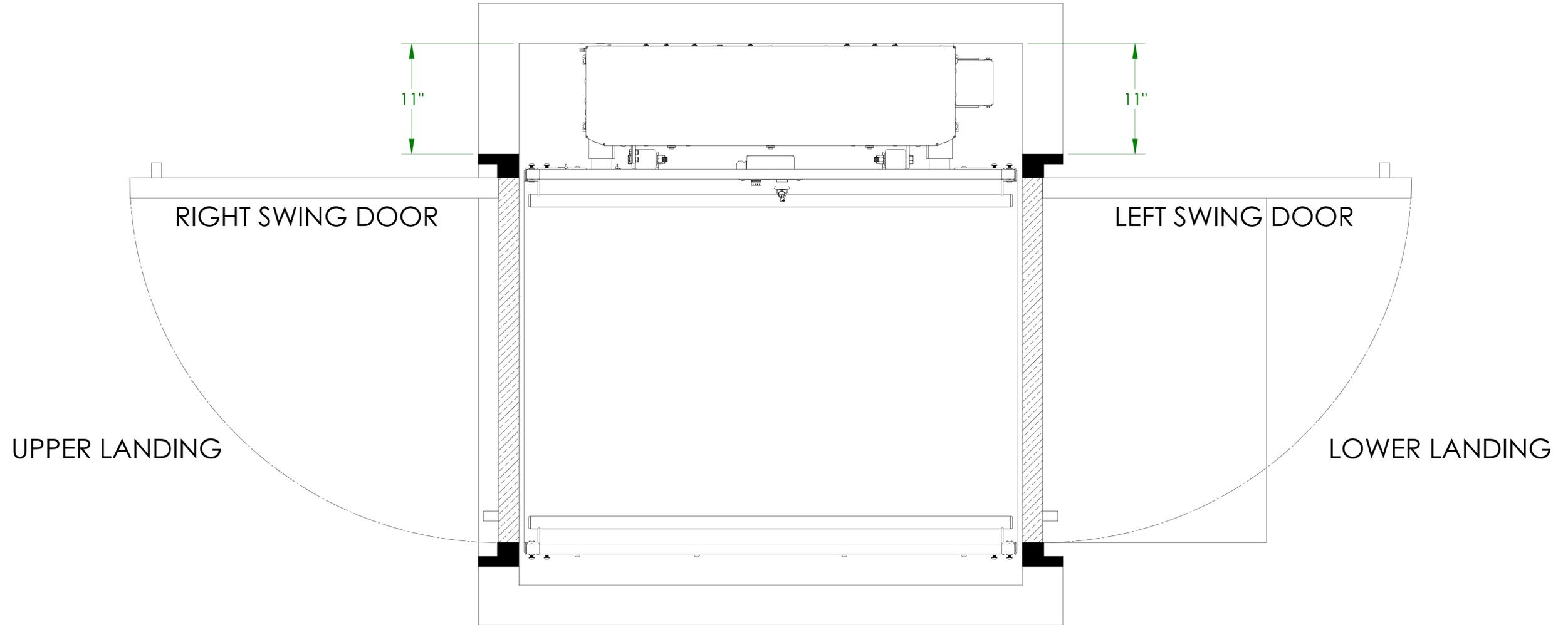

PLAN VIEW
(RIGHT HAND UNIT SHOWN)


ILS-01027
SHEET 2 OF 4
REV. 3 (5917)(6/29/15)(DPG)

FLUSH MOUNT DOOR ALIGNMENT (90° / ADJACENT EXIT PLATFORM) BRUNO VERTICAL PLATFORM LIFT

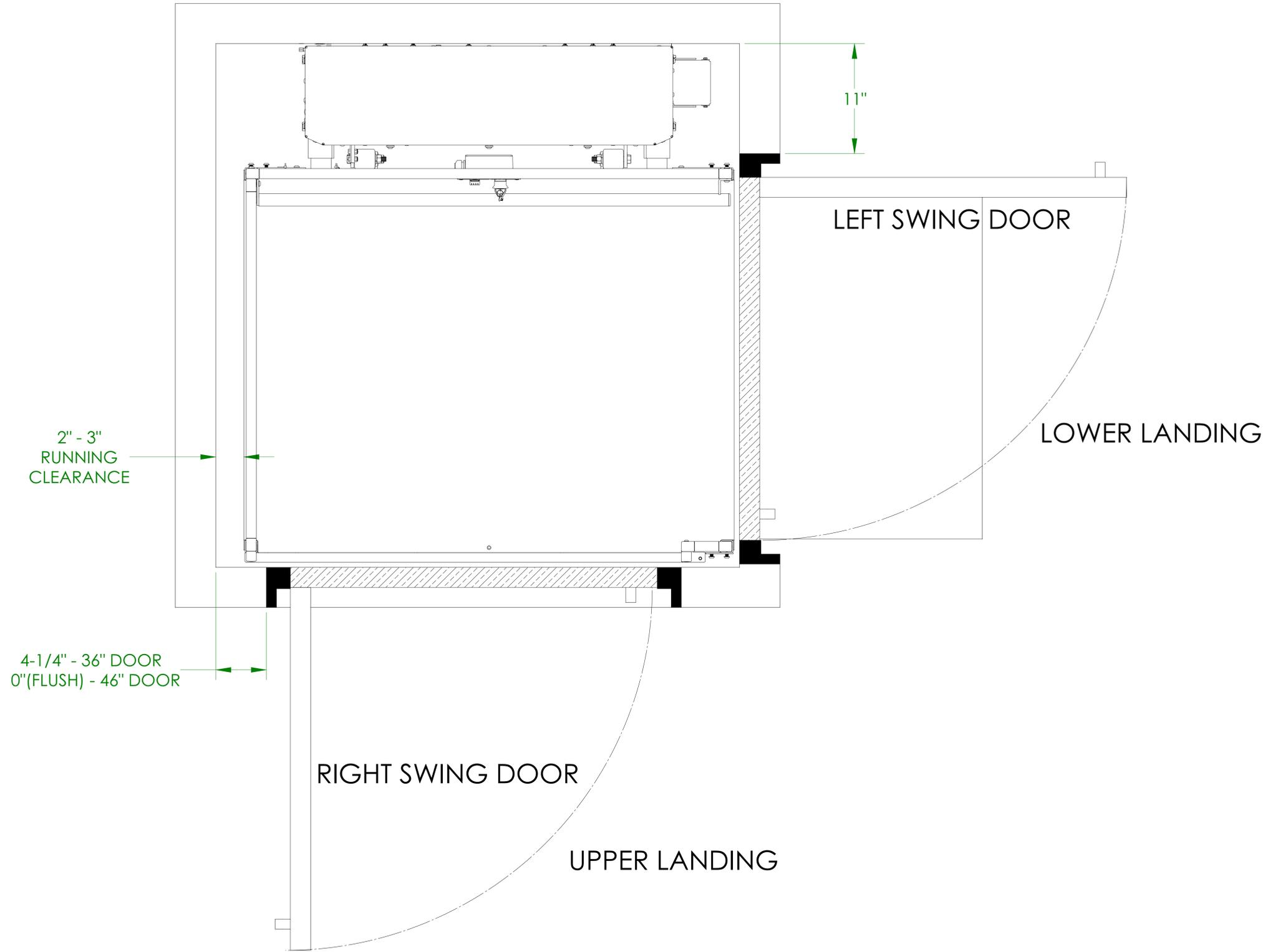

PLAN VIEW
(RIGHT HAND UNIT SHOWN)

FLUSH MOUNT DOOR ALIGNMENT (SAME SIDE PLATFORM) BRUNO VERTICAL PLATFORM LIFT

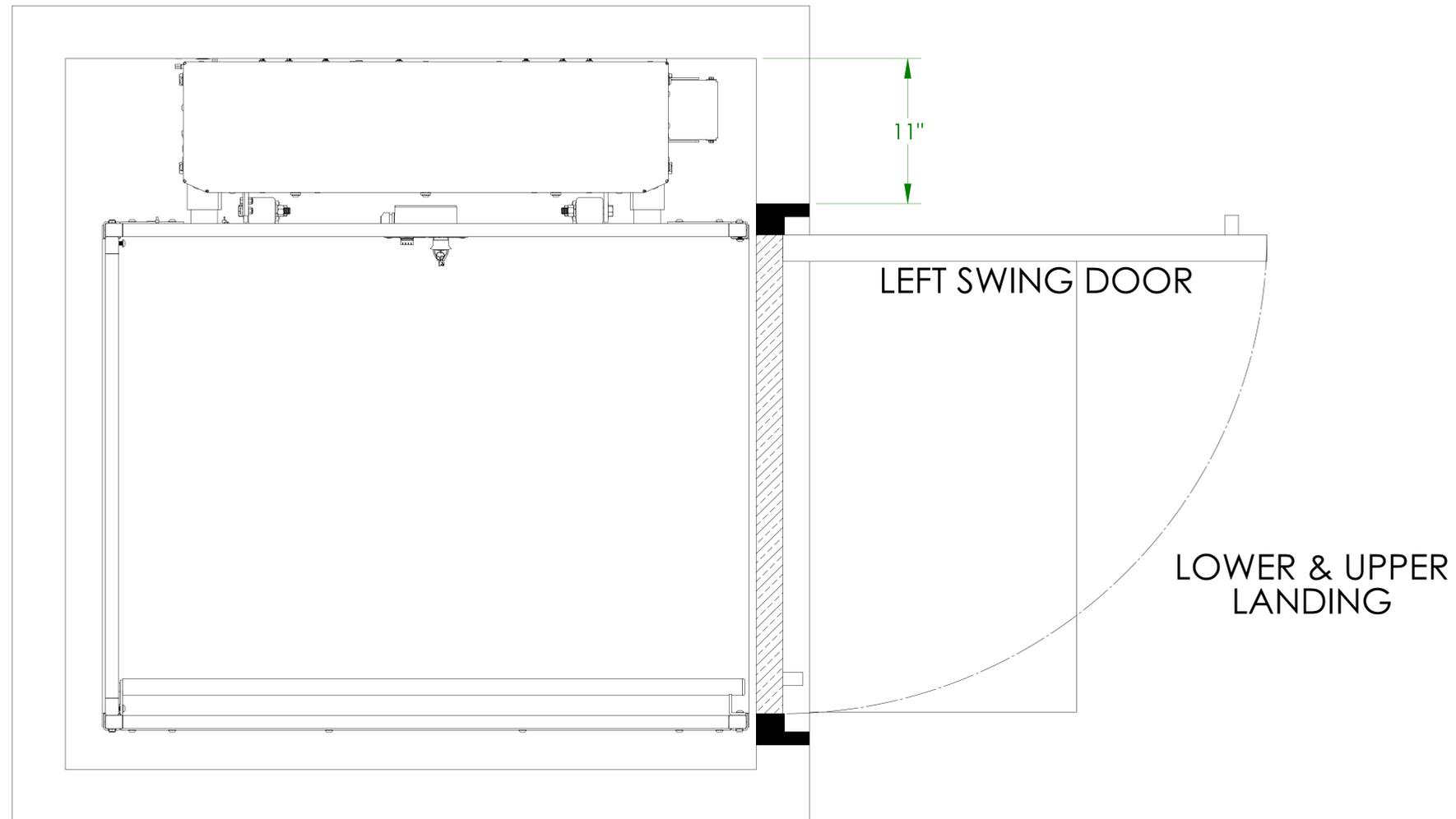

PLAN VIEW
(RIGHT HAND UNIT SHOWN)